

## Explore Chat Editor: Saltillo's Companion Software

### WHAT IS IT? HOW DO I GET IT? WHAT CAN I DO WITH IT?

A FREE companion program for the NOVA chat, Chat Fusion & Liberator Rugged 7, compatible with Windows. Download it from the CD/USB included with the device or online at: <http://saltillo.com/products#chat-editor>

What can I do with it? Why do I want it?

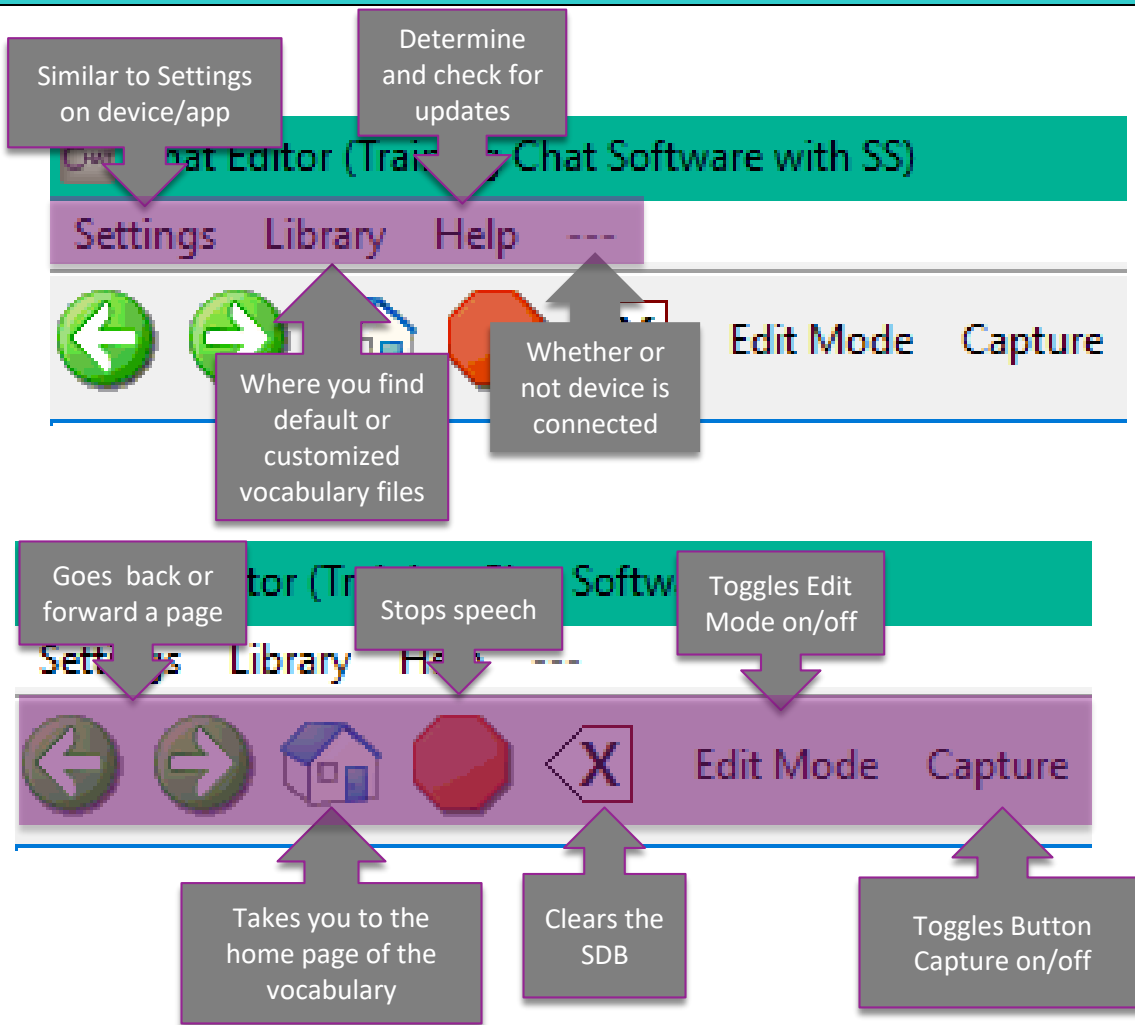


### SETTING UP FOR THE FIRST TIME






You do not need to set up a new configuration every time you connect a new device; however, you will need to switch between a 'Chat' configuration and a 'LAMP Words for Life' configuration if you want to change between these vocabulary options.

## NAVIGATING CHAT EDITOR



### HOW DO I...

Determine the <b>software version</b> of my Chat Editor	<ul style="list-style-type: none"> <li>• Select <b>Help &gt; About</b></li> <li>• Note: If you have an internet connection you can “check online for new version”</li> </ul>
Determine the <b>software version</b> of NOVA chat or Chat Fusion	<ul style="list-style-type: none"> <li>• Select menu   </li> <li>• Select <b>Help &gt; About</b></li> </ul>
<b>Edit</b> on Chat Editor	<ul style="list-style-type: none"> <li>• You must be in a copy in order to edit. You cannot edit original files.</li> <li>• Select the “Edit Mode” button.</li> <li>• Right click on your mouse to bring up the editing menu.</li> </ul>
<b>Customize</b> a Button	<ul style="list-style-type: none"> <li>• Enter <b>Edit Mode</b></li> <li>• Right click on button to modify &gt; Select <b>Edit Button</b></li> <li>• Modify: <ul style="list-style-type: none"> <li>▪ Label: words on the button</li> <li>▪ Message: words spoken</li> </ul> </li> <li>• Image: picture on the button</li> </ul>
<b>Import Pictures</b> directly to a button	<ul style="list-style-type: none"> <li>• Enter <b>Edit Mode</b>, Right click on button to edit &gt; Select <b>Edit Button</b>.</li> <li>• Under “Image” select “Import”</li> <li>• Locate and select desired picture on your computer.</li> <li>• Select OK</li> </ul>

Use the <b>Next Tool</b>	<ul style="list-style-type: none"> <li>• Right click on button to be edited</li> <li>• Select <b>Edit Button</b> &gt; make desired changes</li> <li>• Select NEXT to edit the next button in the row.</li> </ul>
<b>Highlight</b> or change the border color of a button	<ul style="list-style-type: none"> <li>• Enter <b>Edit Mode</b>, Right click on button to edit &gt; Select <b>Edit Button</b>.</li> <li>• Select “Style” tab</li> <li>• Select “Border” &gt; make desired changes to color</li> <li>• Increase Border width from the drop down menu.</li> <li>• When you finish making changes, choose <b>Save</b>.</li> </ul>
<b>Hide</b> or show multiple buttons at once	<ul style="list-style-type: none"> <li>• Enter <b>Edit Mode</b> &gt; select <b>Hide Mode</b></li> </ul> <p>You have three options:</p> <ul style="list-style-type: none"> <li>▪ Choose buttons one by one to hide or show</li> <li>▪ Choose <b>Hide All</b> to hide all buttons on the current page.</li> <li>▪ Choose <b>Show All</b> to show all buttons that had previously been hidden.</li> </ul> <p>Note: You must hide each page at a time. To exit Hide Mode, click on <b>Hide Mode</b> again.</p>
<b>Transfer/Back up</b> files to/from <b>NOVA chat</b> or <b>Chat Fusion</b>	<ul style="list-style-type: none"> <li>• Attach device to computer via USB cable</li> <li>• On Chat Editor: <b>Library &gt; Transfer &gt; Vocab to/from Device</b></li> <li>• Select desired file and desired destination (e.g., To Editor or To Device) &gt; <b>DONE</b>.</li> <li>• Files are added to “My Resources”</li> </ul>
<b>Duplicate</b> a file on Chat Editor	<ul style="list-style-type: none"> <li>• In Chat Editor, select <b>Library</b> &gt; and locate the file you want to copy. Depending on your configuration you may have to open a language (e.g., English), a symbol set (e.g., SymbolStix, PCS), or a vocabulary suite (e.g., WordPower).</li> <li>• Click on desired file &gt; <b>Duplicate</b></li> <li>• Enter a New Name (e.g., WordPower 60_Dave)</li> <li>• File is added to “My Resources”</li> </ul>
<b>Email</b> a customized File	<ul style="list-style-type: none"> <li>• Open previous editor &gt; <b>Library &gt; Transfer &gt; Vocab to File</b></li> <li>• Select file &gt; Save to Desktop</li> <li>• Compose email and attach file</li> <li>• Person receiving downloads/saves file in their Chat Editor folder.</li> </ul>
<b>Print Pages</b>	<ul style="list-style-type: none"> <li>• In Chat Editor Select <b>Settings &gt; Print Pages</b></li> <li>• Select the pages you want to print.</li> <li>• Click on “Print Selected Pages”</li> </ul>
<b>Capture Sequences</b>	<ul style="list-style-type: none"> <li>• In Chat Editor Select <b>Capture</b></li> <li>• Select the desired button sequences. These will appear in a window below.</li> <li>• Select “Copy to Clipboard.” You can paste where desired.</li> </ul>

### DEVICE AND EDITOR DIFFERENCES

- Voice output is only activated on Editor once a device is connected
- Vocabulary Builder cannot be used on Chat Editor
- Settings do not transfer from Chat Editor to NC
- Word Finder is not available on Chat Editor

### OTHER LINKS/RESOURCES

Make a <b>Low-Tech Communcation Book</b> :	<a href="http://saltillo.com/chatcorner">http://saltillo.com/chatcorner</a>
Watch Client and How to Videos on <b>YouTube</b> :	<a href="https://www.youtube.com/user/saltillocorp">https://www.youtube.com/user/saltillocorp</a>
Learn Vocabulary with <b>Practice Sequences</b> :	<a href="http://saltillo.com/chatcorner">http://saltillo.com/chatcorner</a>
Watch Chat Software <b>Video Tutorials</b>	<a href="http://saltillo.com/videos">http://saltillo.com/videos</a>
Watch TouchChat <b>Video Tutorials</b>	<a href="https://touchchatapp.com/support/videos">https://touchchatapp.com/support/videos</a>