

Accent® 800, 1000, and 1400

Quick Reference Guide

Applies to NuVoice® Software Version 2.13 or higher


Liberator Support: (08) 8211 7766

info@liberator.net.au

www.liberator.net.au

YouTube video tutorial playlists

www.youtube.com/user/PRCaccess/playlists

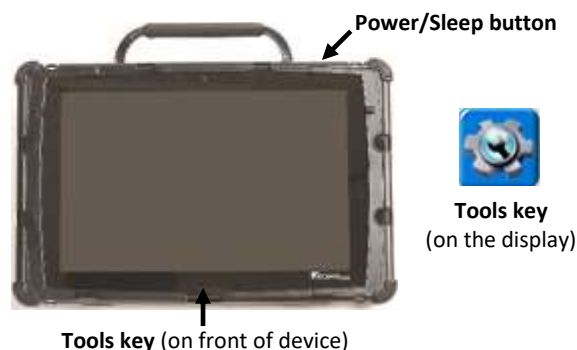

Most of the operations covered on this guide pertain to all language systems: Unity®, UNIDAD®, LAMP Words for Life®, CoreScanner™, WordPower™, and Essence®. Exceptions are noted.

Tools Key (pictured above)

The Tools key reveals a set of User Keys, such as the Toolbox, Shutdown and Setup key. The Toolbox and Setup keys are commonly used.



POWER/SLEEP BUTTON

The power/sleep button is on the top of the device.

1. To power on the device, press and hold the power/sleep button.
2. To wake/sleep the device, press the power/sleep button.
3. The device should be restarted weekly.*

4. To restart, select **Tools, Toolbox, Maint Menu, Power Management, Restart**, and then **Yes**.

*** Recommended: Restart once per week; put to sleep daily. The device may wake up from sleep if using NuEye® or if Windows® updates are set to install automatically. To preserve the charge overnight, either connect the charger or shut down the device completely. See step 5.**

5. If you don't plan to use the device, want to store/transport it in a bag or receive an error message, shut down the device completely. To do this, select **Tools, Shutdown**, and **Yes**.

CHANGING USER AREAS

1. Select **Tools**, and then **Toolbox**.
2. Select **User Area Menu**. Select **Switch User Area**.
3. Select the User Area (1-6) to switch to and wait until the device returns you to the User Area Menu screen. Select **OK**.
4. From the toolbox, press **Go to Home** (blue key).
5. Please note that **Replace User Area** provides more user area choices and will *overwrite* an existing user area.

CHANGING THE VOICE

1. Select **Tools**, and then **Toolbox**.
2. Select **Speech Menu**.
3. Select **Change Voice**.
4. Select the desired voice. Wait for it to load. To listen to the voice, select **Test Speech**.
5. Use the additional keys to adjust the pitch, rate, etc. of the voice. Select **OK** and **Go to Home**.

PRONUNCIATION DICTIONARY

"Teach" the device to pronounce a word correctly.

1. Select **Tools**, and then **Toolbox**.
2. Select **Pronunciation Dictionary**.
3. Choose either **Main Voice** or **Prompt Voice**.
4. Select **Add Pronunciation**.
5. Enter the correct spelling of the word. Select **OK**.
6. Enter the "phonetic spelling" of the word. To test the pronunciation, select the message window.
7. Select **OK**, and then **OK**.
8. From the toolbox, press **Go to Home** (blue key).

CHANGING SIZE OF LABELS

1. Select **Tools**, and then **Toolbox**.
2. Select **Feedback Menu**.
3. Set **Label Size** to Small, Medium, Large, or Largest.
4. Select **OK** and **Go To Home**.

TOOLBOX RESTRICTIONS

There are two ways to restrict access to the toolbox. Either with a **password** or a **USB flash drive**.

To restrict the toolbox with a password.

1. Select **Tools**, and then **Toolbox**.
2. Select **Maint Menu**.
3. Select **Toolbox Restrictions**. First set Toolbox Restrictions to **ON**.
4. Set Lock Type to **Password**.
5. Select **Create Toolbox Password**. Enter a password, then select **OK**. Remember your password.
6. Select **OK, OK**, and then **Go to Home** (blue key).

To unlock the toolbox with a password:

1. Select **Tools**.
2. Enter your password and select **OK**.
3. Select **Toolbox**.
4. Select **Maint Menu**.
5. Select **Toolbox Restrictions** and turn Toolbox Restrictions **OFF**.
6. Select **OK, OK**, and then **Go to Home** (blue key).

To restrict the toolbox with a USB flash drive.

1. Select **Tools**, and then **Toolbox**.
2. Select **Maint Menu**.
3. Select **Toolbox Restrictions**. First set Toolbox Restrictions to **ON**.
4. Set Lock Type to **USB**.
5. Select **Create USB Lock Override**. Insert a USB flash drive into the device. An override file will be placed on the flash drive. The device will "unlock" anytime this USB flash drive is inserted into the device. Select **OK**.
6. Select **OK, OK**, and then **Go to Home** (blue key).
7. Remove the USB flash drive.

To unlock the toolbox with a USB flash drive:

1. Plug the USB flash drive with the override file into the device.
2. The toolbox will unlock.

Additional options

1. You can **enable/disable** the **Hardware Volume Button** and/or the **Tools/Home Key** on the frame of the device (bottom/middle).
2. You can **enable/disable** the **Left, Right** and/or **Bottom Off-screen Keys**.
3. You can **enable/disable** the **On-Screen Tools Key, Help Button**, and/or **Pulldown Menu Button**.

WORD FINDER

In Unity or LAMP Words for Life, use Word Finder to locate a word in the vocabulary.

From the keyboard:

1. Select **Word Finder**.
2. Type the word you want to find.
3. Select **OK**.
4. Select **Guide Me**.
5. If the word is not in Unity or LAMP WFL, it will tell you “*no matches found*”.

TEMPORARILY LIMITING VOCABULARY

Use Vocabulary Builder to temporarily limit the number of words appearing on the screen. For example, you only want 8 words when first introducing the device to a child.

VOCABULARY BUILDER

For Unity 1-hit: [Video](#)

For Unity Sequenced: [Video](#) and [Quick Reference Guide](#)

For LAMP WFL: [Video](#) and [Quick Reference Guide](#)

EDITING A KEY

1. Be sure you are looking at the key you want to edit. Select **Tools**, and then **Setup Key**.
2. Touch the key you want to edit.
3. Select **Change Icon**. Then do one of the following:
 - A. **Spell Icon to Find**. Spell what you are looking for. Select **OK**. Touch the icon you want.
 - B. Select **Take Picture**. Frame picture, then select **Take Picture**. Select **Save as Icon**. **Rename Icon**. Select **OK**.
 - C. Plug a USB flash drive with photos on it into the device. Select **Import Icon**. If the content of the flash drive is showing, touch the photo you want to import. If not, select **Up Folder** to navigate to the flash drive, and then touch the photo you want. Select **OK** and **OK**. Select **Cancel**. The photo will be placed on the key and in the imports folder of the icon dictionary.
 - D. Search for an icon by category.
4. Look to see if the “text to speak” is correct. If not, select **Spell Message or Define Key Function** to type text that will appear in the display and be spoken. Be sure to put a *space* after the last character. Select **OK**.
5. Check to see if the label is correct. If not, select **Change Label**, type new label and **OK**.
6. Select **OK** or **Choose Next Key to Define**.

TAKING A PHOTO

1. Select **Tools**, and then **Toolbox**.
2. Select **Camera Menu**.
3. Frame the picture. Select **Take Picture**.
4. Do either of the following (most common **B. Save as Icon**):
 - A. Select **Save as Scene** to use this photo for a visual scene that spans multiple keys. Select **Rename Scene**, and then select **Clear Display**. Type a name, select **OK**, **OK**.
 - B. Select **Save as Icon** to use this photo for an icon on a single key. Select **Rename Icon** and then select **Clear Display**. Type a name and select **OK**. Select **OK** again. The photo will automatically be placed in the Imports folder. *Note:* When changing an icon, your photos will be in the category called Imports.
5. Select **OK** and then select **Go to Home**.

CREATING A NEW ACTIVITY

(Unity and UNIDAD language systems ONLY)

1. Select **Tools**, and then **Toolbox**.
2. Select **Create Activity**.
3. Spell the name of the new activity. Select **OK**.
4. Select an icon for the activity.
5. Touch a blank key in the activity. A red box will appear around the key. Edit the key.
6. When finished, select **OK** (or touch the next key to be set up). **Go to Home** (blue key).

HIDING ACTIVITIES

(Unity and UNIDAD language systems ONLY)

1. Select **Tools**, then **Setup Key**.
2. Select the activity you wish to “hide”.
3. Choose **Hide** on the right side of this screen.
4. If you don’t see the activity you need, scroll through by selecting **More Items**.
5. When you are finished, select **OK**.

CREATING and LINKING PAGES

1. Be looking at the key that will link to the new page. Select **Tools**, then **Setup Key**.
2. Select the key that will link to the new page.
3. Select **Change Icon** to search for an icon.
4. Select **Spell Message or Define Key Function**.
5. Select **Clear Display**. Select **Page Link**. Decide if you want your page to automatically close after a key is selected (“close”) or remain open (“open”).

A. Find and select the page you want to link if it is already created.

B. If the page is not already created, select **Create New Page**, type the name of the page, and select **OK**. Choose a keyboard size (to format a new page like an already existing page, select **Create Page from Template**). Select **OK**.

6. Select **Change Label** to change the label. Labels for page links are typically in all CAPS in Unity.
7. Select **OK**.

VOCABULARY BACK UP

1. Insert your flash drive into a USB port.
2. Select **Tools**, and then **Toolbox**.
3. Select **Transfer Memory Menu**.
4. Select **Save One User Area** and then the user area you want to back up or **Save Entire Device Contents**.
5. Make sure the pre-set folder is a USB drive. If so, select **OK**. If not, select **Choose different folder** and navigate to the USB drive. Then select **OK**. Type the file name and date (xx-xx-xx).
6. Select **OK** and wait for the memory transfer to complete. Select **OK**, **OK**, and then **Go to Home**.
7. To load a backup file:
 - a. Follow steps 1-3 and then choose **Load One User Area** or **Load Entire Device Contents**. This process will replace what currently exists in the user area or entire device.
 - b. If you are loading one user area, choose a user area to load into.
 - c. Select the backup file on the USB drive.
 - d. Wait for the memory transfer to complete. Select **OK**.
 - e. If asked “Switch to this User Area now?” answer yes or no. Then **OK** and **OK**.

INTERNET SOFTWARE UPDATES *

1. The device must be connected to a Wi-Fi® network.
2. Select **Tools**, and then **Toolbox**.
3. Select **Maint Menu**, and then **Software Update Menu**.
4. Select **Internet Update** and follow the prompts.

* Internet updates only apply to versions 1.08 and higher.