



Your NOVA chat: First Steps

NOVA chat 5 · 2nd Generation
NOVA chat 8 · 1st Generation

NOVA chat 10 · 3rd Generation
NOVA chat 12 · 1st Generation



1 Before You Turn on Your NOVA chat Device

You received a battery charger with your NOVA chat device. Plug the charger's connectors into the NOVA chat device and amplifier. Plug the other end of the charger into a wall outlet.



NOVA chat 5



NOVA chat 8



NOVA chat 10



NOVA chat 12

See "NOVA chat 12 Detail"

NOVA chat 12 Detail

NOVA chat 12 device USB charging port (with back of device facing up)



If you insert a USB 2.0 Micro cable (with the back of the device facing up), the connector will insert into the left side of the port. It will *not* fill the entire port.



If you insert a USB 3.0 Micro B cable, the connector will fill the entire port.




2 Call for Training

Call a speech and language pathologist or a Saltillo consultant for detailed training and guidance with your device. For a list of consultants in your area, go to <http://saltillo.com/reps>.

3 Identify the Buttons on Your Device


You will frequently use the power, volume, menu, home, and back buttons.

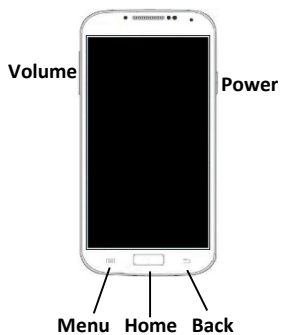


NOVA chat 5



NOVA chat 8

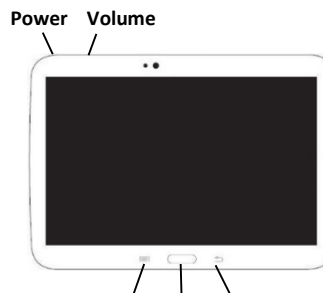

NOVA chat 10

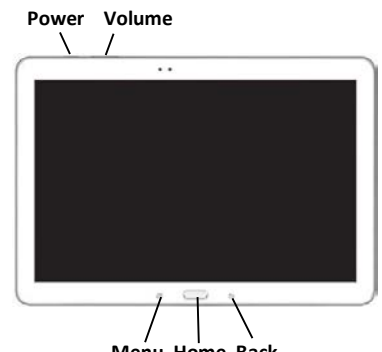

NOVA chat 12

4 Choose a Vocabulary File

If you want to use a different vocabulary file than the one that is currently active, you can select an alternative. Your speech and language specialist may assist in choosing the best vocabulary.


1. Tap on the **Menu** button (☰).
2. Tap on **Library**.
3. Scroll up and down the list to view the available vocabulary files.
4. Select the file you want.




5 Choose a Voice

If you want to use a different voice than the one that is currently active, you can choose from a variety of voices.

1. Tap on the **Menu** button (☰).
2. Tap on **Settings**.
3. Select **Speech Output**.

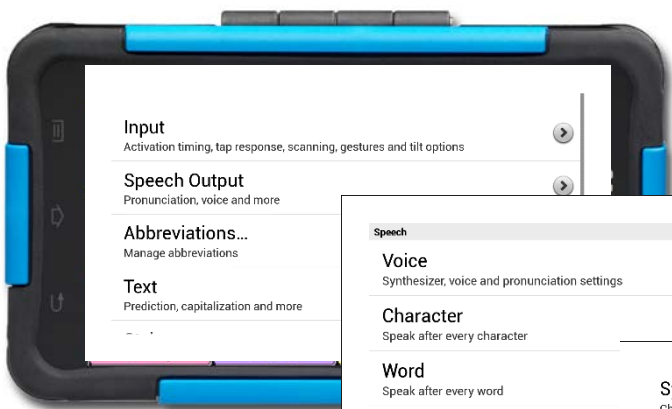


4. Tap on **Voice**.*

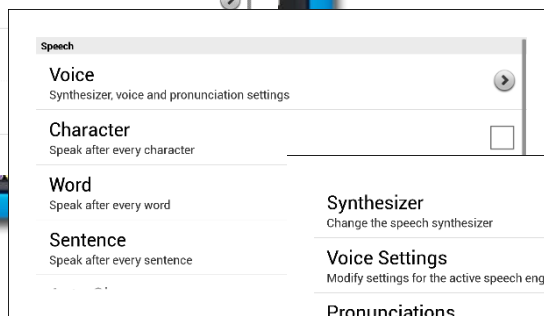

** If you purchased Acapela Synthesizer:*
After you tap on **Voice** (step 4), tap on **Synthesizer** and select **Acapela**. Then proceed to step 5.

5. Tap on **Voice Settings**.

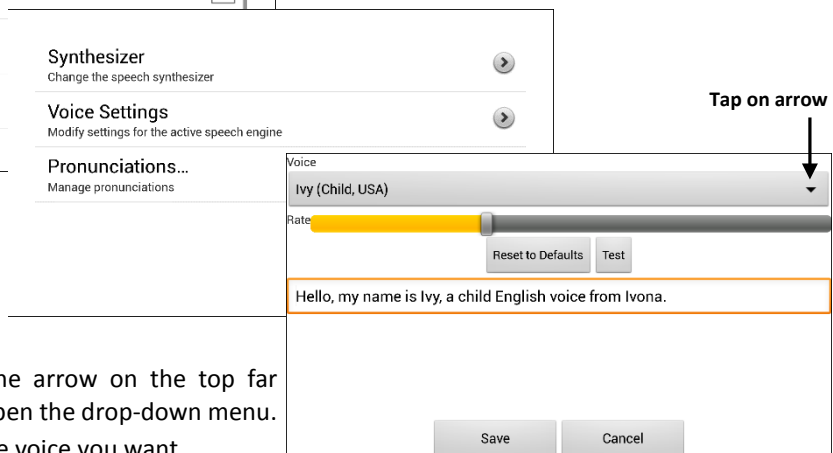

6. Tap on the arrow on the top far right to open the drop-down menu.
7. Tap on the voice you want.

8. Tap on **Save**.
9. Tap on the **Back** button until you return to the vocabulary page.

6 Now, Start Exploring Your Device! Refer to the User's Guide for Assistance.