

Setting up an Accent Device with LAMP WFL or Unity

Device Manuals: [Accent 800](#) / [Accent 1000](#) / [Accent 1400](#)

YouTube Playlist: https://www.youtube.com/watch?v=19Ir6nGISdo&list=PLUhS3sOCHK5x_zqMQeoKzF7euhMRWf147

NB: The videos may show different systems (e.g., Unity, LAMP WFL). The guiding steps remain the same regardless of the system.

Out of the Box – Setting up your Accent device

Video link

EXTRA TIPS & INFORMATION


- [Replace user areas](#): to load different vocabularies, e.g., Exploration Wizard (UNITY Programs > Exploration Wizard).
- To turn labels on/off:
 - Tools key > TOOLBOX > FEEDBACK MENU > Show Category Labels > toggle to 'Both'

- Remember:

Labels tell you about a single word, e.g.,



Symbols (multi-meaning) tell you about **all** the words that are linked by that button as the first hit, e.g.,

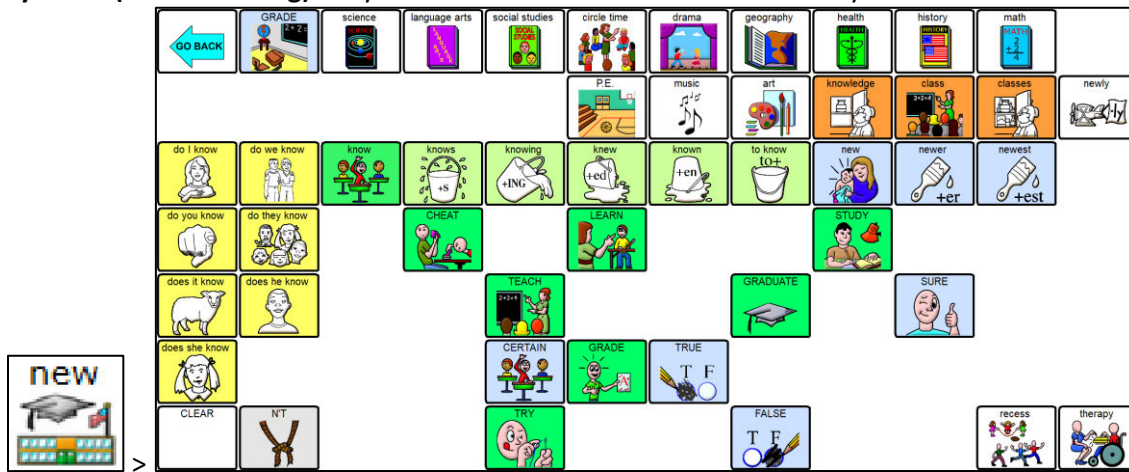

Important Information about Adding Words and Pages

Choosing where to add words and pages to LAMP WFL is best done with your Speech-Language Pathologist, especially when you are first starting out with a device. Here are some videos that can help your team make these decisions:

- [LAMP Words for Life Overview](#)
- [Where to store words when using LAMP Words for Life](#)
- [Introduction to Unity](#) (go to timestamp 19:19 – 27:00 for specific information about word placement)

Both LAMP WFL and Unity are similar systems to one another because LAMP WFL was created from Unity, which is a MinSpeak system. You can learn more about MinSpeak on their [website](#).

Add or edit a button

[Video link](#)

EXTRA TIPS

- You can [take a photo when setting up a button](#) (this video shows a different vocabulary to LAMP WFL and Unity, but the same steps apply).

Add a page

[Video link](#)

Add a Verb or Adjective

[Video link](#)

Similar steps can be used to add adjectives to the system, and for adding verbs/adjective to Unity.

Unity One Hit / LAMP WFL One Hit & Transition

LAMP WFL comes in three versions:

- **LAMP WFL 84 Full**
- LAMP WFL 84 Transition
- LAMP WFL 84 1-Hit

Unity comes in two versions:

- **Unity Sequenced (full)**
- Unity 1-Hit

1-Hit and Transition versions of the systems are designed to be **steppingstones to the full system** and are not adequate systems for robust communication.

If you think that you or the device user who you are supporting might benefit from these, you can load them into a user area (as above). I would recommend having a copy of each in three different user areas so that you can move up quickly when needed.

Vocabulary Builder

A tool that temporarily hides words, allowing focus on a small set of target words while keeping the same consistent motor patterns for those words. It's recommended that these words are agreed upon with your Speech-Language Pathologist.

- [Using Vocabulary Builder](#)
- [Save a Sequence Set in Vocabulary Builder](#)

