

Explore Chat Editor: Saltillo's Companion Software

WHAT IS IT? HOW DO I GET IT? WHAT CAN I DO WITH IT?

A FREE companion program for the NOVA chat, Chat Fusion & Liberator Rugged 7, compatible with Windows. Download it from the CD/USB included with the device or online at: <http://saltillo.com/products#chat-editor>

What can I do with it? Why do I want it?

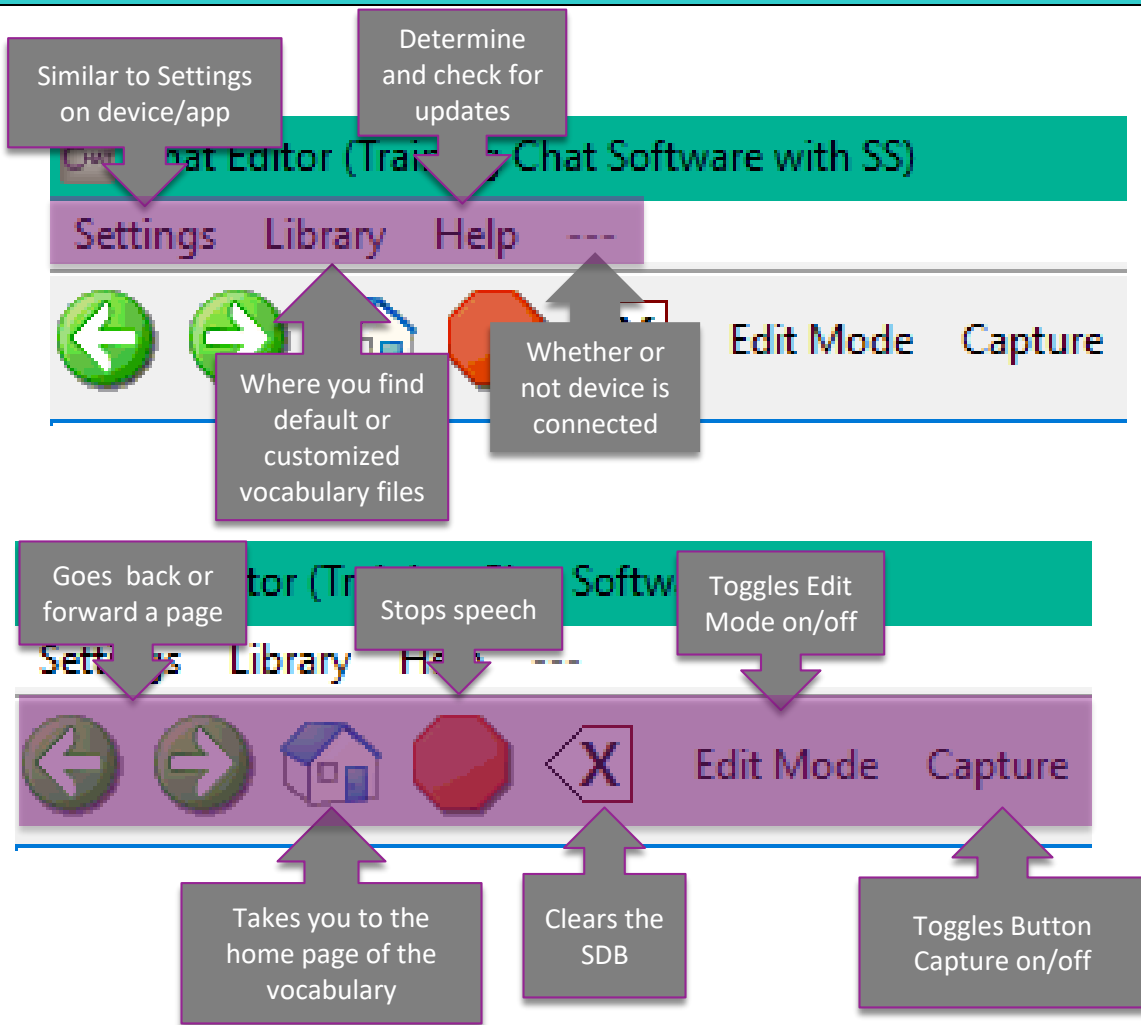


SETTING UP FOR THE FIRST TIME






You do not need to set up a new configuration every time you connect a new device; however, you will need to switch between a 'Chat' configuration and a 'LAMP Words for Life' configuration if you want to change between these vocabulary options.

NAVIGATING CHAT EDITOR



HOW DO I...

Determine the software version of my Chat Editor	<ul style="list-style-type: none"> • Select Help > About • Note: If you have an internet connection you can “check online for new version”
Determine the software version of NOVA chat or Chat Fusion	<ul style="list-style-type: none"> • Select menu    • Select Help > About
Edit on Chat Editor	<ul style="list-style-type: none"> • You must be in a copy in order to edit. You cannot edit original files. • Select the “Edit Mode” button. • Right click on your mouse to bring up the editing menu.
Customize a Button	<ul style="list-style-type: none"> • Enter Edit Mode • Right click on button to modify > Select Edit Button • Modify: <ul style="list-style-type: none"> ▪ Label: words on the button ▪ Message: words spoken • Image: picture on the button
Import Pictures directly to a button	<ul style="list-style-type: none"> • Enter Edit Mode, Right click on button to edit > Select Edit Button. • Under “Image” select “Import” • Locate and select desired picture on your computer. • Select OK

Use the Next Tool	<ul style="list-style-type: none"> • Right click on button to be edited • Select Edit Button > make desired changes • Select NEXT to edit the next button in the row.
Highlight or change the border color of a button	<ul style="list-style-type: none"> • Enter Edit Mode, Right click on button to edit > Select Edit Button. • Select “Style” tab • Select “Border” > make desired changes to color • Increase Border width from the drop down menu. • When you finish making changes, choose Save.
Hide or show multiple buttons at once	<ul style="list-style-type: none"> • Enter Edit Mode > select Hide Mode <p>You have three options:</p> <ul style="list-style-type: none"> ▪ Choose buttons one by one to hide or show ▪ Choose Hide All to hide all buttons on the current page. ▪ Choose Show All to show all buttons that had previously been hidden. <p>Note: You must hide each page at a time. To exit Hide Mode, click on Hide Mode again.</p>
Transfer/Back up files to/from NOVA chat or Chat Fusion	<ul style="list-style-type: none"> • Attach device to computer via USB cable • On Chat Editor: Library > Transfer > Vocab to/from Device • Select desired file and desired destination (e.g., To Editor or To Device) > DONE. • Files are added to “My Resources”
Duplicate a file on Chat Editor	<ul style="list-style-type: none"> • In Chat Editor, select Library > and locate the file you want to copy. Depending on your configuration you may have to open a language (e.g., English), a symbol set (e.g., SymbolStix, PCS), or a vocabulary suite (e.g., WordPower). • Click on desired file > Duplicate • Enter a New Name (e.g., WordPower 60_Dave) • File is added to “My Resources”
Email a customized File	<ul style="list-style-type: none"> • Open previous editor > Library > Transfer > Vocab to File • Select file > Save to Desktop • Compose email and attach file • Person receiving downloads/saves file in their Chat Editor folder.
Print Pages	<ul style="list-style-type: none"> • In Chat Editor Select Settings > Print Pages • Select the pages you want to print. • Click on “Print Selected Pages”
Capture Sequences	<ul style="list-style-type: none"> • In Chat Editor Select Capture • Select the desired button sequences. These will appear in a window below. • Select “Copy to Clipboard.” You can paste where desired.

DEVICE AND EDITOR DIFFERENCES

- Voice output is only activated on Editor once a device is connected
- Vocabulary Builder cannot be used on Chat Editor
- Settings do not transfer from Chat Editor to NC
- Word Finder is not available on Chat Editor

OTHER LINKS/RESOURCES

Make a Low-Tech Communcation Book :	http://saltillo.com/chatcorner
Watch Client and How to Videos on YouTube :	https://www.youtube.com/user/saltillocorp
Learn Vocabulary with Practice Sequences :	http://saltillo.com/chatcorner
Watch Chat Software Video Tutorials	http://saltillo.com/videos
Watch TouchChat Video Tutorials	https://touchchatapp.com/support/videos