



Avaz

Quick Reference Guide

This quick reference guide will help you get started with Avaz.



Setting up the basics

Grid, theme and language

To get started, **register your app** using your email address. Choose the US language to access all grid sizes.

To set up your **grid** choose **Menu > Settings > Picture Grid Settings**

To set up your **theme** choose **Menu > Settings > Theme**

To change your app **language** choose **Menu > Settings > Languages > Change Language**

Voice, navigation and security

To set up your voice:

1. Choose **Menu > Settings**
2. Then select **Voice**
3. You can either select Default/Premium Voice OR Configure Voice Speed and/or Pitch

For **expressive voice tones**:

1. Choose **Menu > Settings**
2. Then select **Voice > Voice > Expressive Voices**

To **stop your app from speaking functional keys**:

1. Choose **Menu > Settings**
2. Then select **Voice > What to Speak**
3. Then select **Speak Words/speak only** message box

To set up your side navigation menu:

1. Choose **Menu > Settings**
2. Then select **Picture Grid Settings > Change Side Navigation Bar**
3. Select from options.

To **set up your favourite folder in the side navigation menu**:

1. Choose **Menu > Settings**
2. Select Picture **Grid Settings > Change Side Navigation Bar**
3. Then select **Favourites > Change Folder**

To **set up a password** (and lock editing / customisation):

1. Choose **Menu > Settings**
2. Then select **General > Password Protection > Set Password**

Backups

To **create and manage backups**:

1. Choose **Menu > Settings**
2. Then select **Backup & Restore**

To **automatically sync backups**:

1. Choose **Menu > Settings**
2. Then select **Backup & Restore**
3. Select **Securely save your backup to the cloud**

For auto backup:

1. Choose **Menu > Settings**
2. Then select **Manage Auto Backups**
3. Select **Turn on > Set Interval**

Words

To **add a new word**:

1. Choose **Menu > Edit Words**
2. Select **+** on grid
3. Select **Word**

To **create a new word folder or category**:

1. Choose **Menu > Edit**
2. Then select **+** on grid
3. Then select word **Folder**

To add multiple words at once:

1. Choose **Menu > Edit**
2. Then select **+Add New**
3. Then select **Many Words at Once**

To edit/delete existing words:

1. Choose **Menu > Edit**
2. Then select the word you want to edit
3. Select **Change/Hide/Delete**

Icons

To **hide/unhide icon image**:

1. Choose **Menu > Edit**
2. Select the word you want to change
3. Then select **> Change > Show Picture / Hide Picture**

To **add custom images**:

1. Choose **Change / Change Picture**
2. Then select from location

To **set a GIF loop count**:

1. Choose **Menu > Edit**
2. **Tap** on your chosen word
3. Then select **Change > Change Picture**
4. Then press **Select GIF > GIF Loop Count**

To **set a YouTube video**:

1. Choose **Menu > Edit**
2. **Select** your chosen word
3. Select **Change > Advanced (top)**
4. Then select **Actions (left) > Add Actions > YouTube**

Getting around quickly

To **automatically return to the previous folder**:

1. Choose **Menu > Edit**
2. Then **double tap on folder**

To **automatically return to the home page**:

1. Choose **Menu > Settings**
2. Select **Picture Grid Settings**
3. Then select **Toggle ON Auto Home** each time (under Behaviour in Picture Settings)

Messages

To **share a message from the message box**:

1. Choose **Construct a message > Tap the Share icon on message box**

To **auto clear message box**:

1. Choose **Menu > Settings**
2. Select **Message Box > Auto Clear Message Box**

Need more help?

Link to our online resources.



ONLINE RESOURCES

Contact us

Our Customer Service staff are available Monday to Friday for general information, technical support, or queries.

(02) 9124 9945 | support@liberator.net.au

liberator.net.au   

Managing your folders

To **link an existing folder to any pageset**:

1. Choose **Menu > Edit**
2. Then select **+Add New > Link to an existing folder**

To **hide selected words in a linked folder**:

1. Choose **Menu > Edit**
2. Select the words you want to hide
3. Then select **Hide (top panel)**
4. Select **Only Here**

To **hide new words in a frozen row folder** when in a particular pageset:

1. **Choose a frozen row folder from pageset** (e.g. tap on **Actions From Food**)
2. Select **Menu > Edit**
3. Then select the words you want to hide
4. Then select **Hide (top panel)** and then **Only Here**

To **change the grid size of a folder**:

1. Choose **Menu > Edit**
2. Then **double tap on folder**
3. Select **Folder Options (top right) > Pictures per screen**

Keyboard hints

To save pre-made phrases on keyboard:

1. Choose **Keyboard > Construct a message**
2. Select **Save (bottom left)**
3. Then select **Choose an existing category / Create New Folder**

To **change prediction settings**:

1. Choose **Menu > Settings**
2. Then select **Keyboard > Prediction**

To **change keyboard layout**:

1. Choose **Menu > Settings**
2. Then select **Keyboard > Layout**

To **highlight vowels**:

1. Choose **Menu > Settings**
2. Select **Keyboard > Layout**
3. Then toggle **Highlight Vowels**