

Changing the Coloured Wrap

NovaRugged™ 8 · 1st Generation



NovaRugged™ 8.1

The wrap provides extra protection for your NovaRugged™ device. It was made to fit tightly and stay on once attached. This makes it a little harder to attach. You will need to stretch the wrap to fit it over the device.

Step 1: Remove the Old Wrap

1. Pull back on the corners of the wrap.



Hint: The wrap has two slits on the bottom.

To get started, hold the wrap on one side of a slit and pull the corner away from the other side (as shown here).

2. Pull around the edges of the wrap to gradually separate it from the device.
3. Remove the wrap from the device.

Step 2: Attach the New Wrap

1. Align the cutouts at the top of the wrap with the device's Volume button, Side key, and Active key at the top of the device.
2. Pull the edges of the wrap over the edges of the device.



3. Press all edges of the wrap firmly into place over the edges of the device.
4. Check all around the edge of the device to make sure that the wrap fits tightly.